

iCup[®] 2

Drug Screen



Key Feature:

“No Step Operation With Split Specimen”

Once the specimen is added to the cup, screwing on the cap initiates the test. No keys or activation steps are required.



iCup[®] 2 Multi-Drug Test Cup

Drug Testing Made Simple.

- Integrated 7 Parameter Adulteration Screening
- Available in 5 - 9 Drug Combinations
- Screw Top Lid Eliminates Leaking
- No Keys or Activation Steps
- Split Specimen Sampling
- All Testing Components Included in Kit

Brands You Can Trust[™]



iCup[®] 2

Drug Screen


iCup[®] 2 Multi-Drug Test Cup

Drug Testing Made Simple.

Detect the presence of drugs and ensure specimen validity in minutes. It's drug testing efficiency you can count on.

Integrated testing solution:

- iCup[®] 2 with temperature strip
- Security Seal
- Two-part result form available
- No keys or activation required

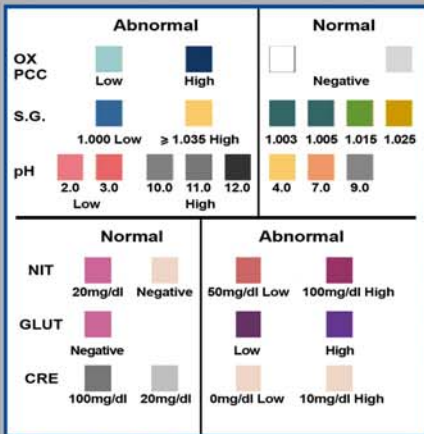

Drug Name:	Abbreviation:	Cutoff Levels:
Amphetamine.....	AMP.....	1000ng/ml
Barbiturates.....	BAR.....	300ng/ml
Benzodiazepines.....	BZO.....	300ng/ml
Cocaine.....	COC.....	300ng/ml
Marijuana.....	THC.....	50ng/ml
Methadone.....	MTD.....	300ng/ml
Methamphetamine.....	mAMP.....	1000ng/ml
Opiates.....	OPI.....	2000ng/ml
Phencyclidine.....	PCP.....	25ng/ml

Sample Only- Color chart NOT Exact
7 parameters on 6 reactive pads

Instructions

- Donor provides specimen (Cap is held by operator).
- The Cap should be screwed on gently but firmly, taking care to properly align the Cap with the threading on the cup while completely depressing the Plunger Rod.
- Temperature is read between 2-4 minutes to verify range between 90-100°F.
- Check cap for tight seal. Donor dates and initials the security seal and places security seal over cap as shown.
- Remove the Peel Off Label covering the test results.
- Read the adulteration strips between 3 and 5 minutes.
- Read results.
 - 5 minutes / 60 minutes: Read the drug strip results at 5 minutes. Test results stable up to 60 minutes.
 - 5 minutes / 60 minutes: Read the drug strip results at 5 minutes. Test results stable up to 60 minutes.
- If preliminary positive results are observed, send the cup to the laboratory for confirmation.

Adulteration Strips: See color chart and package insert for interpretation