

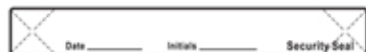


iCup[®] 2 Drug Screen Procedure Card

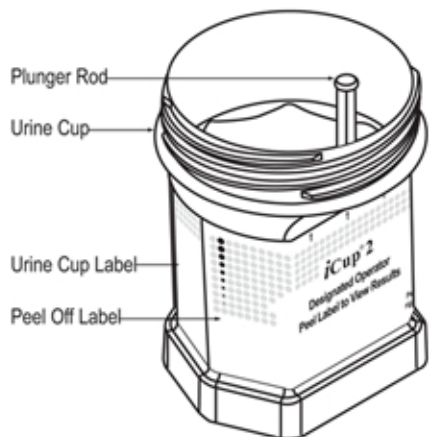

Security Seal

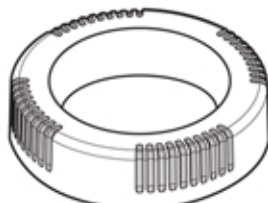


Cap (Included in the foil pouch)


1 Donor provides specimen (Cap is held by operator).



2 The Cap should be screwed on gently but firmly, taking care to properly align the Cap with the threading on the cup while completely depressing the Plunger Rod.

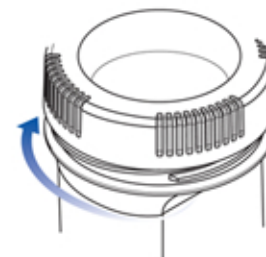


3 Temperature is read between 2-4 minutes to verify range between 90-100°F.

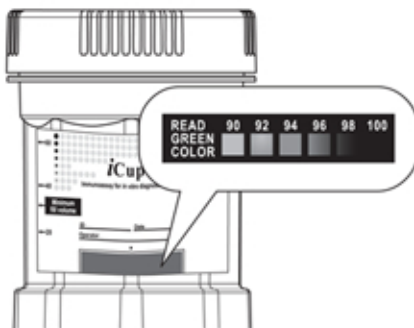


4 Check cap for tight seal. Donor dates and initials the security seal and places security seal over cap as shown.



5 Remove the Peel Off Label covering the test results.

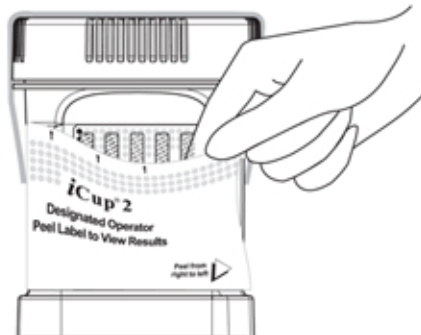


6 Read the adulteration strips between 3 and 5 minutes.

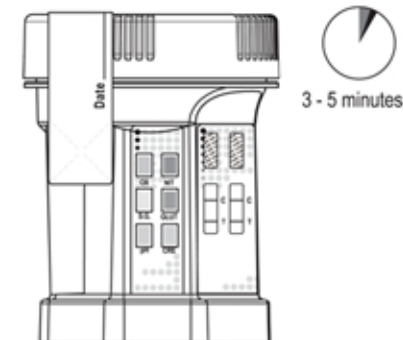


■ Adulteration Strips: See color chart and package insert for interpretation


7 Read results.

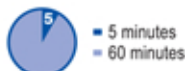

Read the drug strip results at 5 minutes.
Test results stable up to 60 minutes.

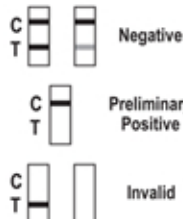

Read the drug strip results at 5 minutes.
Test results stable up to 60 minutes.

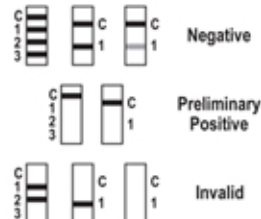

8 If preliminary positive results are observed, send the cup to the laboratory for confirmation.

